BECKEN

[you can trust]

Electric Breast Pump

*Bomba Eléctrica Sacaleches
Bomba de Peito Elétrica*

INSTRUCTIONS MANUAL

MANUAL DE INSTRUCCIONES

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Dear client,

Thank you for purchasing this product.

The quality of this appliance has been certified by several laboratory tests. This instructions manual is provided to simplify the use of the appliance as well as to guarantee a safe use.

Please read carefully the instructions and safety measures described in this manual.

We sincerely hope you will be satisfied when using your new appliance.

INDEX

1. SAFETY INSTRUCTIONS	4
1.1. Power Supply	5
1.2. Power Cord and Other Cables	5
1.3. Humidity and Water	5
1.4. Cleaning	6
1.5. General Precautions	6
2. OPERATING INSTRUCTIONS	7
2.1. Product Description	7
2.2. Using the Product	7
2.3. Maintenance	12
2.4. Troubleshooting	13
3. POST-SALE SERVICE	14
4. ENVIRONMENTAL PROTECTION	14

BECKEN

1. SAFETY INSTRUCTIONS

	WARNING	
	RISK OF ELECTRIC SHOCK DO NOT OPEN	
Warning: to avoid the risk of electric shock, do not remove the cover. The interior does not contain any parts that can be repaired by the user. Ask for assistance from qualified personnel.		

Important Warnings

Before using your appliance read this instructions manual carefully and keep it in a safe place as you may need to consult it in the future.

Before turning on your appliance, and to ensure a safe and correct operation, do not use if:

- It has fallen to the ground;
- You notice any evidence of damage;
- Any anomaly occurs during its operation.

To avoid hazards or deterioration of your appliance through incorrect use it is advisable to read these instructions thoroughly.

Your appliance may not be used for other purposes than the ones intended and solely for household use. Any damage resulting from using the appliance outside this scope, any incorrect use, as well as any manual modifications to the product will not be covered and automatically voids the warranty.

Assistance

In case of malfunction, take your appliance to a qualified technician for assistance.

This appliance can be used by children from the age of 8 and by people with reduced physical, sensorial or mental capabilities and/or lack of experience and knowledge if they have been given instructions on using the appliance safely and if they understand the hazards involved. Children may not play with the appliance. Cleaning and maintenance must not be carried out by children without supervision.

These instructions are also available at **www.suporteworten.pt**

1.1. Power Supply

Make sure the main voltage is compatible with the appliance's. If this is not the case please consult the Customer Support Service.

1.2. Power Cord and Other Cables

Do not tangle the cord or pull it to disconnect the appliance. Also, keep the cord away from warm surfaces.

Never use the appliance if it has a damaged cable or socket.

Take it to the Technical Support Service for replacement.

1.3. Humidity and Water

Do not use your appliance in humid places. Do not allow the appliance to become wet under any circumstances as this may be dangerous. To avoid fire hazard or electric shock do not expose your appliance to humidity or water. Furthermore, do not place your appliance under water (e.g. for cleaning).

BECKEN

1.4. Cleaning

Before cleaning, be sure to switch off the appliance and remove the plug from the main power supply. Always allow the unit to cool down before cleaning.

To clean the exterior of the product use a dry soft cloth only.


1.5. General Precautions

- Please follow the instructions in the user manual which help to properly install and operate this product and enjoy its advanced features. Please keep the instructions for future reference.
- Read all instructions before using the product.
- Maintenance and service operations are only to be carried out by authorized professionals.
- Any damage caused by manual modifications to the product is not covered by the warranty.
- If this product is used for other purposes than the ones originally intended, or if it is not correctly connected, the product may become damaged and the warranty will be voided.

2. OPERATING INSTRUCTIONS

2.1. Product Description


- a - Cover
- b - Massage pad
- c - Breast protector
- d - Body of the pump
- e - Flow valve
- f - Baby bottle
- g - Support - Baby bottle
- h - Thread ring
- i - Teat
- j - Cover
- k - Power Adapter
- l - Diaphragm
- m - Motor
- n - Seal ring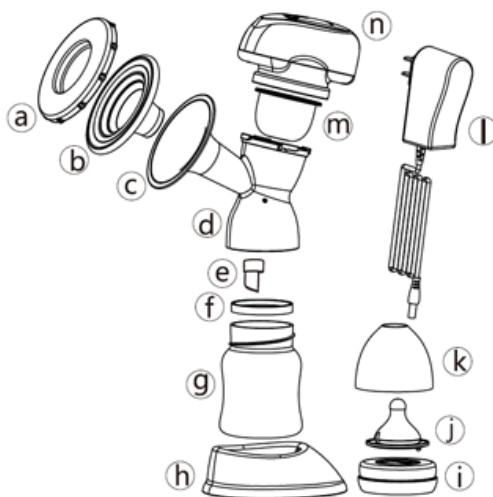

2.2. Using the Product

Control panel description

- 1 - Indicator light
- 2 - On/Off Button
- 3 - Increase button
- 4 - Decrease button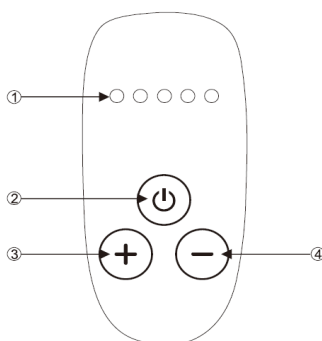

BECKEN


Before first use, disassemble and clean the milk extraction pump.

- Hold the body of the pump using your left hand, with the widest opening facing left. Hold the motor with your right hand. Turn the motor counterclockwise by about 45 degrees to remove it.
- Disassemble the other parts accordingly.


Assembly steps


Make sure you have cleaned and sterilised the appropriate parts of the milk extraction pump. Wash your hands carefully before handling the clean parts.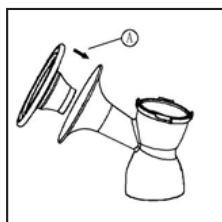


- Insert the massage pad (part A) into the breast protector, securing it to the collar.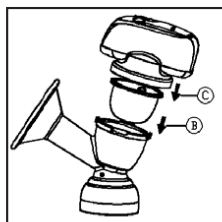


- Place the diaphragm (part C) into the upper body of the pump (part B). Make sure it fits firmly. Press the edge of the diaphragm. Fit the motor into the body of the pump.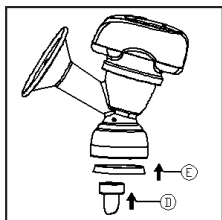


- Firmly insert the seal ring (part E) and the flow valve (part D) into the underside of the body of the pump.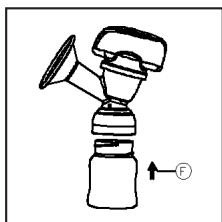


- Tighten the bottle until it is firmly attached (part F) to the pump body.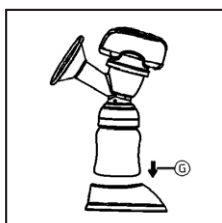

- You can hold the unit by the base (part G).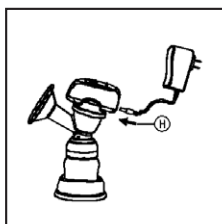

- Connect the power adapter (part H) to the pump motor.

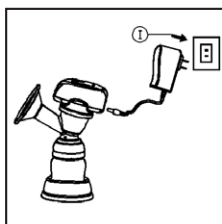- Plug the power adapter to the socket.

BECKEN

Milk pump operation

- It is advisable to find a quiet and comfortable place. Make sure you have a glass of water close to you.
- Wash your hands carefully and make sure your breasts are clean.
- Press the body of the pump against one of your breasts. Make sure your nipple is centred with the breast protector, so that the massage pad creates a vacuum grip.
- Press the  On / Off button to start the operation. If there is no intervention on your part, three consecutive phases will be run automatically.

One light on - massage mode, which lasts 60 seconds.

Two lights on - stimulation mode, which lasts 30 seconds.

Three lights on - normal express mode, the unit continues running until manual intervention.

- Press the  or  button to complete the automated cycle.
- Press the  or  button to increase or decrease the suction and speed level. There are five suction and speed levels.

NOTE: You do not need to use all the suction modes, just use the modes that are comfortable for you.

- Do not worry if your milk does not start flowing right away. Relax and continue to use the pump. The first few times you use the milk pump you may need to use a higher suction mode to start the milk flow.
- Adjusting the correct suction – Depending on your personnel comfort you may want to use a high suction mode that provides stronger suction, or you can return to a lower suction mode.

- After completing the extraction, turn off the milk pump and carefully remove the body of the pump from your breast. To release the vacuum place a finger between the breast and the massage pad.
- Unscrew the bottle from the pump body, it will be ready for feeding / storage.

NOTE: Always turn off the milk pump before removing the body of the pump from the breast.

- Breastfeeding production adjusts to daily requirements. The daily amount required for the baby can vary considerably.
- You should extract milk first from one breast and then from the other. However, the extraction should not exceed 7 minutes for each breast. The following ranges have been shown to be effective, but may vary from woman to woman:

Right 5-7 minutes	Left 5-7 minutes
Right 3-5 minutes	Left 3-5 minutes
Right 2-3 minutes	Left 2-3 minutes

Breast milk storage

Temperature / Storage Location	Recommended Storage Life
19 °C - 26 °C / Room temperature	4 – 6 hours
4 °C Or less / Refrigerator	Up to 24 hours
-18 °C - 20 °C / Freezer	3 - 6 months

BECKEN

- Do not store the breast milk in the refrigerator door. Instead, choose the coolest part of the refrigerator (at the back of the glass shelf, above the vegetable compartment).

If using frozen breast milk, allow it to thaw completely before warming it. To warm the milk we suggest you use a bottle warmer.

2.3. Maintenance

Cleaning and sterilising

The engine and power adapter do not need to be cleaned or sterilized. Clean and sterilize all parts of the milk pump before first use. Clean all parts after each use and sterilize them before the next use.

NOTE: Never place the motor or power adapter in water or in a sterilizer. If necessary, you can clean the engine with a moistened, soft cloth.

- After each use, disassemble the pump to clean. Wash all parts except the engine and power adapter.
- To clean the flow valve and the diaphragm, gently rub them with your hands, under hot water with a little dishwashing liquid. Do not insert objects into the flow valve or the diaphragm as this may cause damage.

NOTE: All maintenance operations must be performed after the device has been unplugged from the power socket.

2.4. Troubleshooting

Try the following solutions in the event that your appliance does not operate as expected. If the milk pump cannot operate normally, contact your dealer, the shop where you purchased the product or the support service.

Problem	Solution
- If the engine does not work	1. Check that the device is powered.
- If suction is weak or non-existent	1. Make sure the massage pad form a full seal around the breast. 2. Check that you have set up the milk pump correctly by referring to the appropriate steps in the “Assembly Steps” chapter.

BECKEN

3. POST-SALE SERVICE

BECKEN has designed this appliance in order to guarantee maximum reliability. However, if a problem occurs do not attempt to open the appliance - there is a risk of electrical shock. If you are not able to solve the problem, please contact the Customer Support Service of any Worten, Modelo or Continente store in order to be assisted. Worten Equipamentos para o Lar provides a 2-year warranty for manufacturing defects from the date of purchase of the appliance.

WARNING: Any attempt to repair the appliance without contacting the manufacturer or seller will void the warranty.

All users must be familiar with these instructions. Knowing what could go wrong can help prevent problems from occurring.

4. ENVIRONMENTAL PROTECTION

In order to protect the environment we strive to reduce the volume of packaging used, limiting it to three easily sorted materials: cardboard, paper and plastic. The appliance is manufactured using recyclable materials after disassembled by a specialized company. Please comply with local regulations concerning the recycling of all materials.

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Worten Equipamentos p/ o Lar S.A.